


DaVinci Pro Quick Start Guide



Quick Start DaVinci Pro

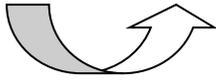

Swivel camera head
to change viewing
target

Tap: Power
on/standby
Press & hold:
Power off

Changes color
contrast

Turn knob to change
magnification +/-

Push to toggle between
OCR & CCTV modes.
This is the only way to
toggle between OCR &
CCTV modes.

DaVinci Pro Quick Start Guide

Basic Mode

Easy to Use

CCTV mode: Press to find/locate

OCR mode: Press to scan/capture

CCTV & OCR modes:

Rotate for

magnification +/-

CCTV mode:

Brightness

OCR mode:

Pause/play

CCTV & OCR

modes: Change

color mode

CCTV & OCR

modes:

Volume

CCTV mode:

Lines & windows

OCR mode:


Reading speed



DaVinci Pro Quick Start Guide

Advanced Mode

Feature Rich

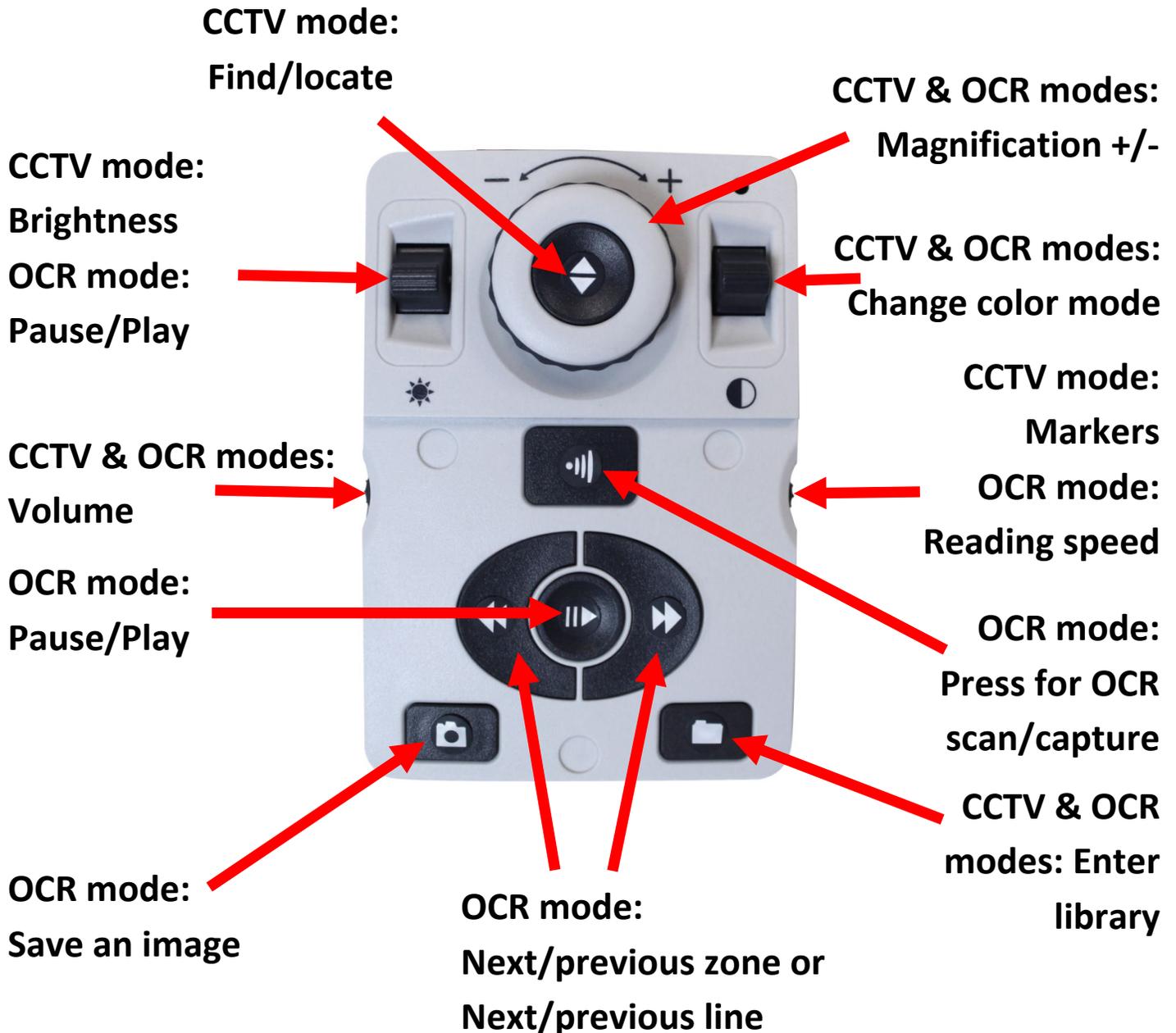


DaVinci Pro Quick Start Guide

Tips and Tricks

Quick Tips

Toggle between CCTV & OCR mode by pressing the center button on the front panel of DaVinci Pro.

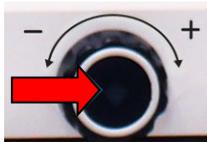

Remove console cover to switch between Basic and Advanced mode.



Configure menu options by pressing Mode Up & Find (center console button).



The figure to the right shows the display and camera adjustments you can make to Davinci Pro for optimal viewing.

